



835 S Highway 105 Suite D
Palmer Lake, CO 80133

Office: 719 622 7200
Fax: 866 299 2495

sterisil.com

IMPORTANT

PROVIDING SAFE WATER FOR YOUR PATIENTS AFTER A 2-WEEK OFFICE CLOSURE

How do you ensure your waterlines are properly treated after closing for 2 weeks?

Prior to closing the dental practice, ensure your office has up-to-date maintenance products in place. For any closures longer than two weeks, purge and air dry the waterlines, empty the bottle and reattach the empty bottle to the unit.

SHOCK THE WATERLINES WHEN YOU GET BACK INTO THE OFFICE:

Citrisil™ Shock Tablets - Part #: ECS-20 (Box of 20 tablets) *For instructions on how to use Citrisil shock, please see page 2.*

- If using **Citrisil™ Tablets** – Shock with Citrisil Shock tablets before using the chair.
- If using the **Sterisil® Straw** – Leave the Straw on the pick-up tube with the bottle reattached, then shock with Citrisil Shock tablets before using the chair.
- If using the **Sterisil® Cartridge** – Run all handpieces for two minutes every seven days to ensure new residual treatment is introduced into the lines.
- If using a **Sterisil® System (Ac, Ac+, G4 or G5)**
 - a. Bottle Fill: Shock with Citrisil Shock tablets before using the chair.
 - b. Direct Feed: Run all handpieces for two minutes every seven days to ensure new residual treatment is introduced into the lines.

To ensure complete waterline compliance, we recommend testing your water with a third-party lab, such as Agenics Labs. **Call 719-466-5592 or visit agenics.net.**

To order more Sterisil product contact your supplier, and for questions related to your office situation notify your local Sterisil representative.

Please visit ADA.org for the most current closure guidelines in your area.

719 622 7200 or visit Sterisil.com/product-guide
Safe, Simple, Sterisil



SHOCKING WITH CITRISIL:

Bottle Fed Dental Units:

- Shock each bottle/line using Citrisil Shock.
- Empty water from bottle and press orange Citrisil Shock tablet into bottle.
- Add 177 ml. (6 fl.oz.) of warm distilled water. Wait 60 seconds for tablet to fully dissolve.
- Swirl orange shock solution to clean the inside of the bottle.
- Connect the bottle to dental unit and run each line until orange color appears.
- Leave in lines overnight.
- The following day empty orange shock solution from bottle.
- Start using a maintenance product to continually prevent bacteria growth.

SHOCKING WITH STRAWS:

If using the Sterisil Straw, keep the Straw installed and perform the shock process through the Straw.